



# Patellavator<sup>®</sup> Knee Orthosis


Before using this product, please carefully read all instructions and warnings.

*Application Instructions*




a.

Detach outer laces from base strap. Place *Patellavator* knee orthosis with Med Spec logo right side up, just below patella (knee cap).

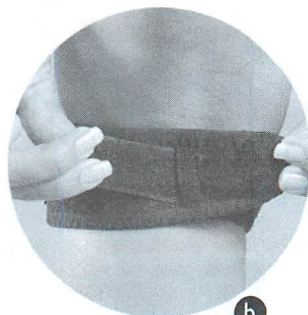

b.

Insert angled strap through slot behind knee to create an interlocking closure. **Pull the strap completely through (no hook fastener material should be next to skin). If uncomfortable to pull too tight, select larger size.**

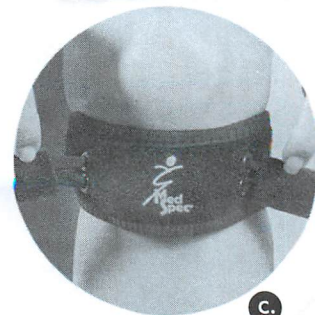

c.

Pull external laces outward until tight and secure to base strap.

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

**Washing Instructions:** Engage all closure straps before washing. Machine wash, warm water, gentle cycle, no bleach, air dry

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

Medical Specialties, Inc.  
4600-K Lebanon Rd.  
Charlotte, NC 28227

p: 800-334-4143 f: 800-694-9060  
p: 704-694-2434 f: 704-694-9060  
e-mail: request@medspec.com

www.medspec.com  
U.S. Patent # 6,852,088

EU Authorized Rep.  
Basko Healthcare  
Pieter Liefinckweg 16  
NL-1505 HX, Zaandam

CE  
rev 2/12