

1. Place patient's arm inside sling,
2. Enclose hand in sling,
3. Apply shoulder strap,
4. Adjust both ends of shoulder strap for comfortable fit.
5. Apply waist strap to brace . Adjust both ends of waist strap. This will keep the sling from rotating.

**Care:** Hand wash using mild soap. Rinse thoroughly. Air dry only . Do not tumble dry.

**Warning:** This device will not prevent or eliminate risk of injury. Do Not Over-tighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.



**Össur Head Office**  
Grjótháls 5  
110 Reykjavík, Iceland  
Tel: +354 515 1300  
Fax: +354 515 1366  
mail@ossur.com

**Össur Americas**  
27051 Towne Centre Drive  
Foothill Ranch, CA 92610,  
USA  
Tel: +1 (949) 382 3883  
Tel: +1 800 233 6263  
Fax: +1 800 831 3160  
ossurusa@ossur.com



*Life Without Limitations®*