




# Ryno Lacer<sup>®</sup>

## Wrist and Thumb Support

Before using this product, please carefully read all instructions and warnings.

### Application Instructions

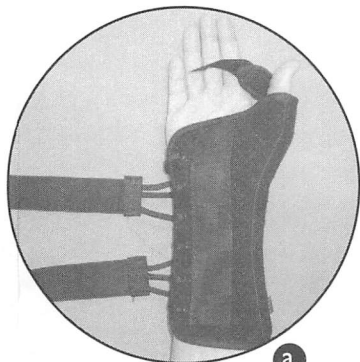


a.

Detach all hook & loop fasteners and slide hand under stockinette liner. Position palmar stay so it matches the curvature of the palm.



b.

Fasten thumb strap.

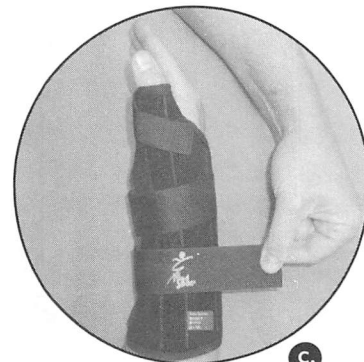

c.

Pull both strap/lace closures tight and fasten to wrist support.

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Medical Specialties, Inc.  
4600-K Lebanon Rd.  
Charlotte, NC 28227  
p: 800-334-4143 f: 800-694-9060  
p: 704-694-2434 f: 704-694-9060  
e-mail: request@medspec.com  
www.medspec.com

**Washing Instructions:** Engage all closure straps before washing. Machine wash, warm water, gentle cycle, no bleach, air dry.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

EU Authorized Rep.  
Basko Healthcare  
Pieter Liefinckweg 16  
NL-1505 HX, Zaandam



rev 2/12